



Fortress Paramount Bobath Neurological 2-Section



Owner's Manual



WARNINGS:

1. *Do not use this device for undiagnosed pain syndromes until consulting a physician.*
2. *Improperly setup, operated or poorly maintained equipment may cause personal injury.*
3. *To avoid this hazard, read and follow the instructions in this manual carefully.*
4. *Check the padding and upholstery when unpacking the table. Do not use it if there are signs of damage. (Our warranty excludes damage caused to the table by improper set-up, accident, abuse, or use other than the table's intended purpose.)*

SAFETY INSTRUCTIONS:

Read the following very carefully and ensure that all staff members who operate the table are fully briefed on safety compliance.

1. *Do not place your hands or feet near the moving parts of the table while adjusting the table sections.*
2. *Use both hands to adjust the angle or the height of any section of the table.*
3. *Do not adjust the section angle when a client is on the table.*
4. *Do not crawl under the table under any circumstances as to avoid injury from a potential table collapse.*
5. *Be certain that the table is properly placed on a flat surface before allowing a client to lie on it; otherwise, the patient may lose balance and fall.*
6. *Do not put excessive weight or pressure on the headrest or table extender.*
7. *Keep children and pets away from the table.*



ELECTRIC SHOCK DANGER:

- 1. The power supply/control box is located below the table. For safety purposes, do not touch the power supply and control box when the table is in operation.*
- 2. Set up the table and complete all the preparatory work before connecting the power.*
- 3. Check the voltage rating on the serial plate of the motor. The plug of the motor should only be inserted into 110 VAC or 220 VAC power outlets as indicated on the serial plate.*
- 4. Do not expose the table to extreme moisture or temperature to avoid shorting the power/control box.*

WEIGHT LIMIT:

(including client and accessories)

Lifting Capacity: 728lbs/330kgs



INSTRUCTIONS FOR USE:

Adjusting The Height of The Table:

- Height adjustment is achieved with footbar switch..



Adjusting The Head Section:

HEAD SECTION

- To raise the head section: operate the head section by manual system (Picture 1). Adjust manual system lever to the “up” side.
- To lower the head section: operate the head section by manual system (Picture 2). Adjust manual system lever to the “down” side.



Picture 1



Picture 2



Applying The Foot Release Caster System:


All Tables are available with a foot release caster system

- To get mobility: place your foot on the positioning lever and push downward (down).
- To get stationary: lift positioning lever up with your foot (up).



Using The Adjustable Foot:

Each table comes with one adjustable foot. It can be adjusted up or down in order to stabilize the table if it is on an uneven surface

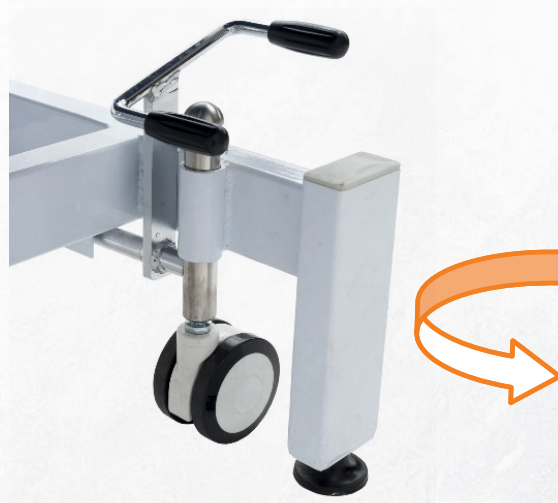

CAUTION:

- Do not use the foot lever system during treatment/ therapy or for patient transport.
- The mobile options is only for table transport itself.



MAINTENANCE:

- *Electrically powered tables are equipped with a long-running, maintenance-free electric motor. Should any problems occur with this unit, please contact us.*

HOW TO CARE FOR YOUR UPHOLSTERY FABRIC:

Here are some steps to protect the table's upholstery and ensure a long life for the product.

- *Avoiding storing the table in extremely cold temperatures since excessive cold will cause the upholstery to crack. Keeping the table in an unheated area during the winter is not recommended.*
- *Avoiding storing the table in extremely warm temperatures since excessive heat will cause the upholstery to stretch and pucker. Keep the table away from heaters, electrical appliances, direct sunlight and the inside of your car during hot weather.*
- *Clean the table's upholstery after each day of use. We recommend cleaning the upholstery with a mild detergent to protect it against oil stains and set-in dirt. A common practice is to use a spray bottle filled with a mixture of anti-bacterial dish soap and water on the upholstery followed by a wipe down with a clean towel.*
- *Protect the table from general abrasions, cuts and punctures.*

CAUTION:

- ***Rubbing alcohol will dry out the upholstery and causing it to crack.***
- ***Moisture may cause the electric actuator to short. Keep the table away from water.***



WARRANTY:

The table comes with lifetime structural warranty, a three-year warranty on frame frame, and a one-year warranty on table upholstery, and "Timotion" motor include three parts, Linear Actuators 5 years, Control box 2 years and foot switch one year. Please note that your warranty certificate is automatically activated upon completion of payment. Therefore, no registration is necessary. However, an original sales receipt or proof of purchase may be required in order to redeem the warranty. This statement of guarantee prevails over any other statement by the seller specified or implied.

THANK YOU FOR PURCHASING THIS TABLE MANUFACTURED WITH HIGH-QUALITY COMPONENTS.



Fortress Equipment

Web: www.fortressequipment.com

Email: info@fortressequipment.com



Copyright 2016